

INSTRUCTIONS FOR USE

CRYOCHAMBERS

OPERATING INSTRUCTIONS

Place the open edge of the Chamber on the surface of the tumor to be treated.

Make certain that the entire 360 degree edge is sealing on the surface.

A more uniform freeze will be achieved if the Chamber is held in a vertical position, therefore, we have included the Malleable Extension to facilitate this.

If the vertical position is maintained, the pool of liquid nitrogen will create an extremely deep symmetrical freeze.

CAUTIONS

The vent tube should be pointed in a safe direction away from the operator and the patient. As the freezing progresses, a wet liquid level line will appear on the outside of the Chamber. This line can be maintained with intermittent pressing of the trigger on your CRY-AC, CRYOGUN or KRYOSPRAY II Unit. It does not have to be filling the Chamber continually. This will require a pressure of 1 to 2 pounds.

We emphasize the depth of 3cm to 4cm of destruction may be achieved, therefore significant caution should be taken

If you have any other questions or comments, please do not hesitate to contact us on (800) 777-CRYO (USA) or 0800 838 274 (UK) or email us at brymill@brymill.com.

EU Authorized Representative –

Brymill Cryogenic Systems (UK) Ltd, 54 Mayfield Ridge, Basingstoke. RG22 4RS. U.K
email: sales@brymilluk.com